

Steps to STERILISATION

1. Clean Instruments

Items **MUST** be thoroughly scrubbed clean

Some instruments require additional cleaning using an ultrasonic cleaner or oxidising biocide



2. Dry and Package Instruments

Instruments **MUST** be thoroughly dried and placed in the package in an open position

The package **MUST** be properly sealed

3. Sterilise in an approved bench-top steriliser

The following **DO NOT** sterilise:

~~UV cabinets, Boiling, Ultrasonic cleaners, Soaking, Pressure cookers, Disinfectants, Pasteurisation, Microwave ovens~~

Load steriliser correctly – **DO NOT** overload
Keep records of each batch

